

Derma-Stent™ is a minimally invasive cutaneous abscess drain

Mar-Med offers the specially designed Derma-Stent as a potential solution for increasingly common cutaneous abscesses.

Effective Healing

Keeps incisions open to allow drainage and healing from within.

Easy to Use

Non-absorbent biocompatible silicone is easy to insert and remove.

Comfortable


Soft and low profile, reducing discomfort for clinicians and patients alike.

Versatile Design

For use with 1 or 2 incisions on abscesses of various sizes.



Low Cost

Similar to the cost of packing gauze.

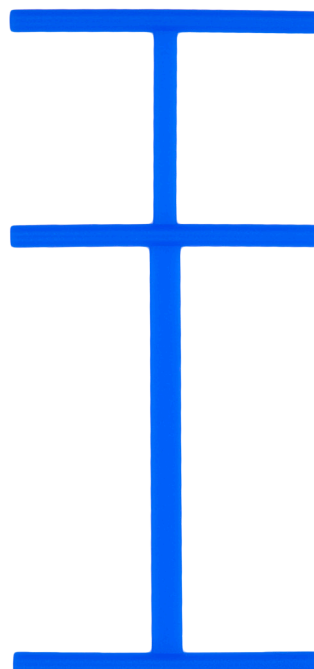

*Actual size

PATENT PENDING

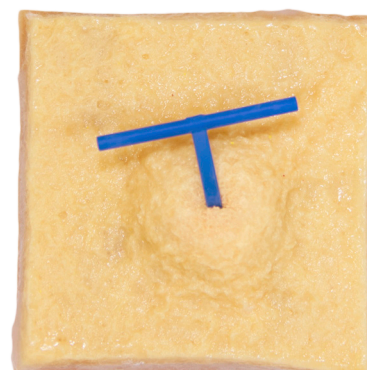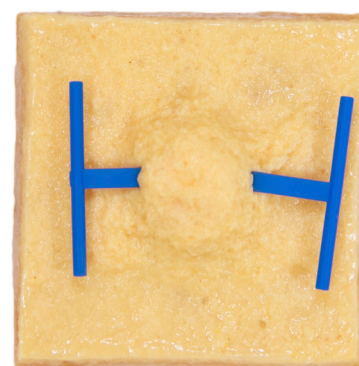


Method/Benefit	Easy to Use	Reduce Visits	Complex* Suitable	Saves Space	Specialty Design
Derma-Stent™	✓	✓	✓	✓	✓
Packing			✓		
Loop Drain		✓	✓	✓	
No Product	✓	✓		✓	

*Complex I&D (CPT Code 10061) = 5.12 RVU, Simple I&D (CPT Code 10060) = 2.77 RVU


Provided solely for informational purposes, is not a guarantee of payment, and should be verified by providers. Each procedure should be reviewed to evaluate best treatment for a patient. No head-to-head studies have been performed to suggest clinical efficacy or superiority.

Derma-Stent™ - Instructions For Use


- 1) Drain abscess using
1 or 2 incisions.

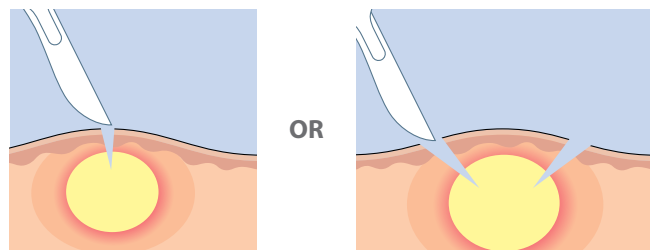


- 2) Measure the abscess along
Derma-Stent. Cut to appropriate size.

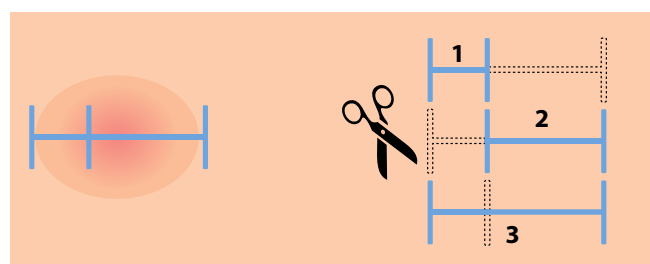

- 3) Insert Derma-Stent into
abscess cavity.

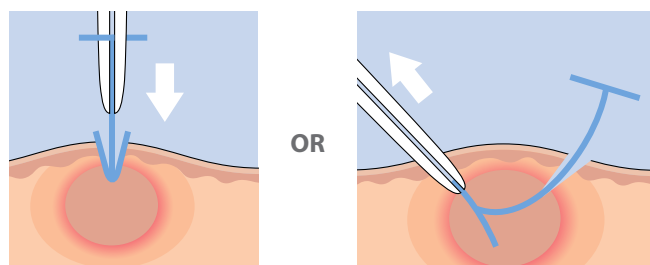

- 4) Remove Derma-Stent in accordance
with clinician instructions.

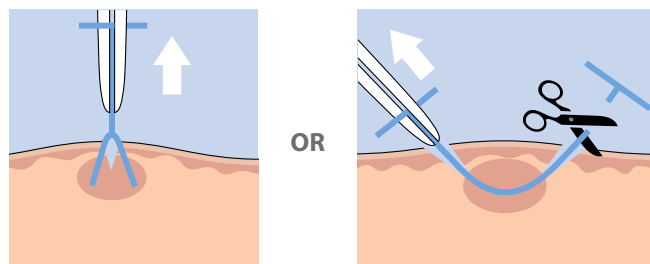

Place Your Order Today

To learn more about the Derma-Stent or to place an order, contact Mar-Med at **1.800.369.3434** or **info@marmedco.com**

MAR-MED

333 Fuller Ave., NE, Grand Rapids, MI 49503 USA

800.369.3434
616.459.5119 Fax

info@marmedco.com
www.marmedco.com